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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read the following instructions to ensure proper use of the product.

Please keep this manual for future reference and follow its recommendations, as failure to follow its instructions may pose a threat to life or health.

APPLICATION AND DESCRIPTION

Teeth grinding and jaw clenching can lead to many serious damages – both to the temporomandibular joint as a whole and to the teeth themselves.

The aligners help stabilize the jaw position and prevent teeth grinding at night.

The duration of wearing relaxation splints depends on the needs of each patient.

The trays can be used to whiten teeth. Special thermoactive trays allow the gel to reach everywhere and work optimally on the entire teeth.

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or if you notice damage due to faulty packaging or transport, please contact the service hotline.

HOW TO USE

- Heat the water to near boiling point.
- Place the overlay vertically in water so that it is fully immersed for about 20-30 seconds. The overlay becomes more transparent. Do not allow its walls to stick together due to plasticization of the material.
- Remove the tray from the water. Before placing it in your mouth, check with your fingers to make sure it is not too hot.
- Place the tray in your mouth as shown in the picture, covering the entire dental arch in a U-shape.
- Close your lips and hold the splint pressed with your fingers, cheeks and tongue for 1-2 minutes while sucking water out of the splint.
- Remove the overlay from the dental arch and cool it under a stream of cold water (approx. 1 minute)
- The overlay can be cut to size, ensuring the necessary comfort of use. Please note that precise adjustment of the overlay reduces gel consumption and ensures optimal use of the product
- Note: If the overlay is too cold to fit your teeth or its walls are stuck together, you can repeat the heating operation.

TECHNICAL DATA

- Heat shrink material
- A universal size overlay
- It is not a dental product
- It is not recommended to eat or drink hot beverages while wearing it.

CLEANING AND MAINTENANCE

- Before each use, rinse the attachment thoroughly under running, lukewarm water.
- After each use, wash the overlay with mild soap or a special oral accessory cleaner.
- Avoid using hot water – it may cause deformation of the material.
- Dry the overlay with a soft cloth or allow to dry on a clean towel.
- Store in a ventilated container, away from moisture and heat sources.
- Regularly check the overlay for damage – if cracked or deformed, replace it with a new one.
- Do not use detergents, alcohol or bleach for cleaning - they may damage the surface of the product.

SAFETY INSTRUCTIONS

- Do not use the overlay if it is cracked, deformed or shows signs of wear.
- Before first use, read the manufacturer's instructions to make sure the product is suitable for your needs.
- Do not share the overlay with other people – the product is intended for individual use only.
- Do not use for longer than recommended time – it may cause discomfort or irritation.
- If you experience pain, discomfort or excessive pressure during use, discontinue use and consult your dentist.
- Keep the overlay away from children and animals - risk of choking or swallowing.
- Do not expose the product to high temperatures (e.g. hot water, sunlight, heating devices).

TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING

The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center.

Used packaging material should be delivered to a waste disposal point designated by local authorities. Information on the possibilities of recycling the used product is provided by the municipal or city office.

The product meets European and national requirements on the safety of devices and products.

We reserve the right to make changes to text, design and product data without notice.

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