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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read the following instructions to ensure proper use of the product.

Please keep this manual for future reference and follow its recommendations, as failure to follow them may pose a risk to life or health.

APPLICATION

A nasal aspirator is a device designed to effectively remove runny nose and mucus from the nasal cavity, especially in infants and young children who cannot clear their noses on their own. It facilitates breathing, improves sleep quality, and helps prevent upper respiratory tract infections. Thanks to its battery power and quiet operation, the device is ideal for both day and night use.

For safety and CE certification reasons, this product must not be rebuilt or modified in any way. Using the product for purposes other than those described may result in damage to the product. Improper use may also result in hazards such as short circuits, fire, electric shock, etc.

ACTIVATION

- Note: Packaging material must be completely removed from the product.
- Make sure the device components are properly assembled.
- Before first use, charge the device using the USB cable. Connect the USB cable to a USB power source and connect the USB cable plug to the product's USB input port.
- Before first use, wash all tips and the secretion container thoroughly in warm water.
- Select the appropriate silicone tip and attach it to the aspirator
- Turn on the device with the ON/OFF button
- Set the appropriate suction level
- Gently insert the tip into the baby's nostril and begin suctioning.
- After completing the treatment, turn off the aspirator, empty the container and clean all components according to hygiene recommendations.
- The device can be used several times a day, depending on the child's needs.

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or damage due to faulty packaging or transport, please contact the service hotline.

TECHNICAL DATA

- Product material: plastic, silicone
- Power regulation: 5 suction levels
- Power supply: battery
- Charging via USB cable
- Filters : replaceable (7 pieces in set)
- Quiet operation

SAFETY INSTRUCTIONS

- This product may be used by children aged 8 years and over and by persons with reduced physical, sensory or mental capabilities or lack of experience or knowledge, provided they are supervised or have been instructed on the safe use of the product and understand the resulting risks.
- Never immerse the device in water.
- For cleaning, use a damp cloth or a mild detergent.
- Do not use a damaged device.
- The product is not intended for use by children.
- Make sure packaging materials are not left unattended. Children may play with them, which is dangerous.
- Protect the product from extreme temperatures, direct sunlight, strong vibrations, high humidity, moisture, flammable gases, vapors and solvents.
- Do not expose the product to mechanical stress.
- If safe operation is no longer possible, discontinue use and secure the product against further use. Safe operation is no longer possible if the product: - has been damaged, - is not functioning properly, - has been stored for an extended period under unfavorable conditions, or - has been subjected to excessive stress during transport.
- Do not use the product if any part is damaged. If the cord is damaged, do not attempt to repair it yourself.
- Do not disassemble the device yourself.
- Protect the product from moisture.

- Make sure not to damage the USB cable with sharp edges or hot objects. Fully unwind the USB cable before use.
- The included USB cable is only suitable for this product.
- Turn off the product immediately and remove the cable from the product if it smells burning or smoke is visible. Have the product checked by a qualified technician before using it again.
- If you're using a USB power adapter, the power outlet must be easily accessible at all times so that you can easily unplug the USB power adapter in an emergency. Please also follow the manufacturer's operating instructions.
- Before each charging process, check the USB cable and device for any damage. Never use or charge the device if you notice any damage.
- Before use, ensure that the existing mains voltage complies with the required operating voltage of the device.
- Do not insert the tip too deep into the nose – gentle contact with the entrance to the nostril is sufficient.
- Use only soft, original silicone tips.
- Clean the aspirator thoroughly after each use and allow it to dry.
- Do not use the device after feeding – it is best to wait at least 30 minutes.
- Replace filters regularly to ensure proper operation and hygiene.
- The product should always be used as intended.

SAFETY INSTRUCTIONS FOR BATTERIES/ACCUMULATORS

- Keep batteries/rechargeable batteries out of the reach of children. If swallowed, seek medical attention immediately!
- Disposable batteries must not be recharged. Batteries/rechargeable batteries should not be short-circuited and/or opened. This may result in overheating, fire, or explosion.
- Never throw batteries/rechargeable batteries into fire or water.
- Batteries/rechargeable batteries should never be subjected to mechanical stress.
- Risk of acid leakage from batteries/accumulators.
- Avoid extreme conditions and temperatures that may affect batteries/accumulators, e.g. radiators/direct sunlight.
- If batteries/rechargeable batteries leak, avoid contact with skin, eyes, and mucous membranes! Immediately flush the affected areas with clean water and seek medical attention!
- Leaking or damaged batteries/rechargeable batteries can cause chemical burns when in contact with skin. Therefore, wear appropriate protective gloves.
- The product has a built-in battery that cannot be replaced by the user.

TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING

The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center.

Used packaging material should be delivered to a designated waste disposal site designated by local authorities. For information on recycling options for used products, please contact your local municipality or city office.

DISPOSAL OF USED ELECTRICAL AND ELECTRONIC EQUIPMENT

For environmental reasons, used electrical and electronic products should not be disposed of with regular household waste, but should be disposed of properly. Information on collection points and their opening hours can be obtained from the relevant authorities.

This product complies with the requirements of the applicable European and national directives. The product meets European and national requirements for the safety of devices and products.

This product complies with the requirements of the relevant European and national RoHS directives.

Improper disposal of batteries/accumulators poses a threat to the environment!

Batteries/accumulators should not be disposed of with household waste. They may contain harmful heavy metals and should be treated as special waste. The chemical symbols for heavy metals are: Cd = cadmium, Hg = mercury, Pb = lead.

Therefore, used batteries/accumulators should be taken to municipal hazardous waste collection points.

The built-in battery must not be disassembled for disposal. The product should be taken to a collection point for used electronics.

We reserve the right to make changes to the text, design and technical data of the product without notice.

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