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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read the following instructions to ensure proper use of the product.

Please keep this manual for future reference and follow its recommendations, as failure to follow its instructions may pose a threat to life or health.

### **DESCRIPTION**

An orthopedic pillow is the perfect solution for people looking for support and comfort during sleep. Thanks to its ergonomic shape, the pillow ensures proper body positioning, reducing tension in the spine, hips and knees. Made of high-quality memory foam, it perfectly adapts to the shape of the body, offering support and comfort at the highest level.

### **APPLICATION**

- Reduces tension in the lower spine.
- Helps relieve hip, knee and back pain.
- Improves body posture during sleep.
- Supports regeneration after injuries and traumas.
- Suitable for pregnant women, providing additional support during sleep.

### **TIP**

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or if you notice damage due to faulty packaging or transport, please contact the service hotline.

### **TECHNICAL DATA**

- Dimensions: 23cm x 22cm

### **SAFETY INSTRUCTIONS**

- The orthopedic pillow is intended for use during sleep or rest. It should not be used as a toy, seat or for other purposes that may damage it.
- Make sure the pillow has no mechanical damage such as cracks, tears or deformations that may affect its effectiveness.
- The pillow is not heat resistant. Keep it away from open flames and heat sources to prevent the risk of fire.
- The pillow should be stored in a dry environment to avoid the development of mold or fungi on the material.
- Regularly clean the removable cover to ensure hygienic conditions. Do not wash memory foam in the washing machine - gently wipe it with a damp cloth if necessary.
- Do not sit or stand on the cushion as this may damage its structure and reduce the functionality of the memory foam.
- If you are allergic to the materials from which the pillow is made, consult your doctor before using it.
- Children should not use the pillow without supervision to avoid the risk of misuse or damage to the product.
- If the outer cover or the foam itself becomes damaged, discontinue use of the product to avoid contact with the inner materials.
- Once you receive your new pillow, leave it in a well-ventilated room for a few hours to eliminate any odor from the manufacturing process.
- Avoid using the pillow in humid environments or on wet surfaces to prevent damage to the materials.
- If you have any spine, hip or knee conditions, consult your doctor before regularly using an orthopedic pillow.

### **STORAGE, CLEANING**

- The pillow should be stored in a dry, airy place, away from moisture to prevent the growth of mold and fungi.
- Do not store the pillow in direct sunlight as this may affect the durability of the memory foam.
- Store your pillow in a protective cover or box to prevent dust from collecting.
- Do not store the pillow near radiators, stoves or other heat sources as high temperatures may damage the foam.
- If you are taking your pillow with you when traveling, use a protective bag to prevent dirt or damage.
- Use a gentle detergent
- Do not wash the wetsuit in a washing machine or immerse it in water as this may damage the material.
- If dirty, gently wipe the surface with a damp cloth or sponge. Avoid excessive wetting.
- Allow the foam to air dry completely in a well-ventilated area.
- When cleaning the foam and cover, avoid using harsh chemicals, bleaches or stain removers that may damage the material.
- To remove odors, expose the pillow to fresh air for a few hours (avoid direct sunlight).
- To remove pet hair or minor dirt, you can use a lint roller or a vacuum cleaner with a soft attachment.
- Wash your pillowcase every few weeks or as needed to maintain hygiene.
- Only clean the foam when necessary.
- Maintaining proper hygiene and proper storage will keep your pillow in good condition for a long time, ensuring comfort and functionality.

### **TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING**

The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center.

Used packaging material should be delivered to a waste disposal point designated by local authorities. Information on the possibilities of recycling the used product is provided by the municipal or city office.

The product meets European and national requirements on the safety of devices and products.

We reserve the right to make changes to text, design and product data without notice.

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