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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read the following instructions to ensure proper use of the product.

Please keep this manual for future reference and follow its recommendations, as failure to follow its instructions may pose a threat to life or health.

APPLICATION AND DESCRIPTION

The elastic band, stabilizing the wrist and hand, has been specially designed to provide support and stabilization of the joints, reducing the risk of injury. Made of high-quality, elastic material that provides a comfortable fit to the hand. Ideal for use while playing tennis, basketball, volleyball and many other sports activities. Excellent for everyday use for people who have problems with strain, sprains of the wrist.

Material composition: 100% polyester

CARE AND CLEANING

To maintain the quality of the material, follow the care instructions below. The product is machine washable according to the manufacturer's recommendations.

MACHINE WASHABLE MAX. 40°C PROGRAM FOR
SYNTHETIC FABRICS

DO NOT DRY CLEAN

DO NOT IRON

DO NOT TUMBLE DRY

DO NOT BLEACH

DO NOT SOAK OR WRING OUT

SAFETY INSTRUCTIONS

- It is advisable to wash the stabilizer before first use.
- Children must not play with the product.
- Cleaning and maintenance may not be performed by children without supervision.
- Make sure the brace fits snugly around the joint. It shouldn't be too loose or too tight to avoid putting pressure on blood circulation.
- Wearing for too long can lead to muscle and joint weakness.
- Dry your stabilizer thoroughly after each use to prevent moisture build-up that could lead to skin irritation.
- Do not use stabilizer on irritated or damaged skin to avoid further aggravation of the skin condition.
- Check the skin under the stabilizer regularly to ensure there is no irritation, redness or abrasions.
- Make sure that packaging materials are not left unattended. Children may start playing with them, which is dangerous.
- The product is not fire resistant, so keep it away from open flames, candles, radiators and other heat sources.
- The stabilizer should be washed according to the manufacturer's recommendations. Too high a washing temperature or the use of inappropriate cleaning agents can damage the material and reduce its durability.
- Wash your stabilizer regularly to avoid skin irritation caused by sweat.
- Store in a dry place to prevent mold and unpleasant odors.
- Always use the product as intended.

TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING

The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center.

Used packaging material should be delivered to a waste disposal point designated by local authorities. Information on the possibilities of recycling the used product is provided by the municipal or city office.

The product meets European and national requirements on the safety of devices and products.

We reserve the right to make changes to text, design and product data without notice.

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