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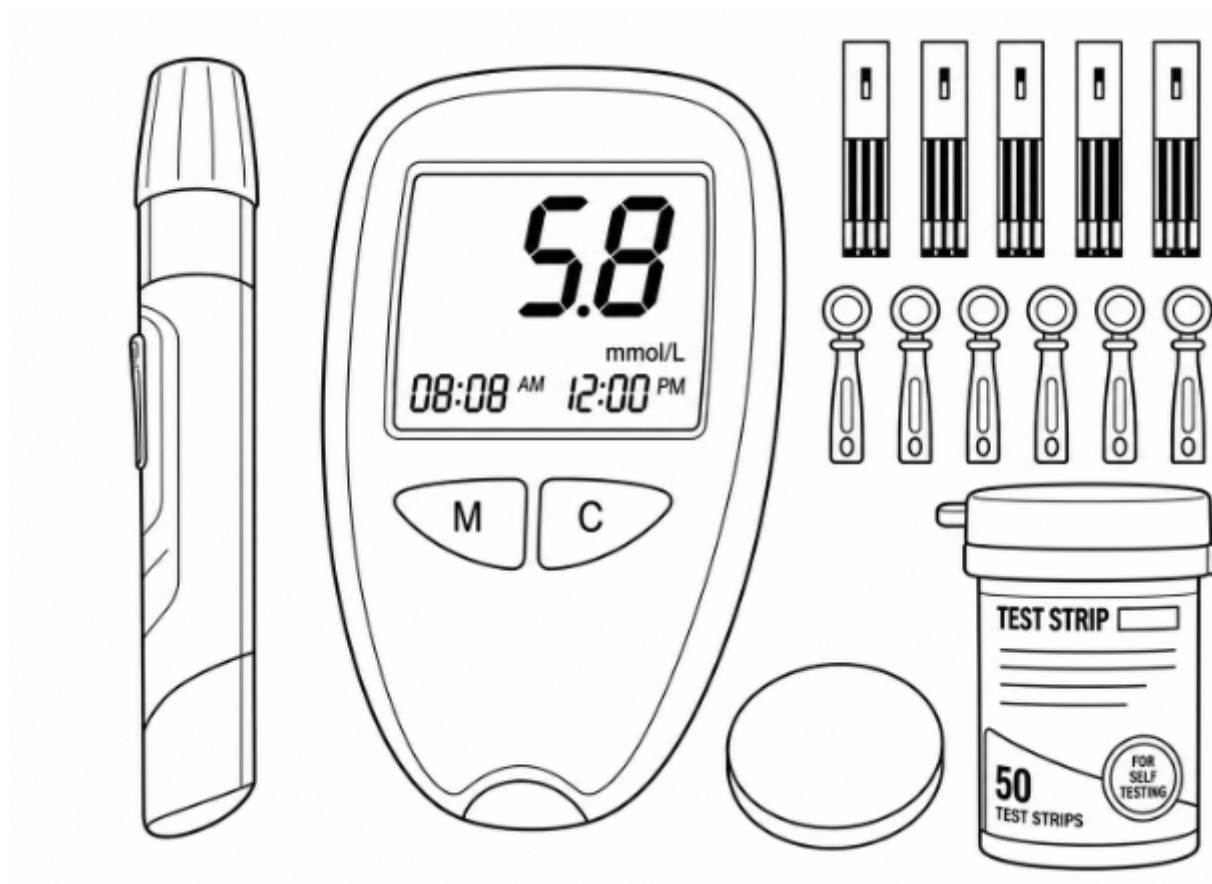
Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read carefully with the instructions below for proper use of the product.

Please keep this manual for future reference and following its recommendations, because failure to follow it may pose a threat to life or health.

Introduction to the user manual

This user manual has been prepared to help you familiarize yourself with the proper and safe use of the device. It contains important instructions on operation, maintenance, and safety rules, which, if followed, will ensure the smooth operation of the device and extend its service life.

Before first use, please read the instructions carefully. The information contained herein will also help you properly store the device, especially if it will not be used for an extended period.

Following the recommendations below will allow you to enjoy trouble-free and safe operation of the product.

ATTENTION!

Using the device in a manner inconsistent with the information contained in the manual may result in the risk of bodily injury.

Application and description of the device

The glucometer is designed for self-measurement of glucose concentration in human capillary whole blood. The device allows for quick measurement at home using a compatible test strip and a small blood sample.

This device is intended for home use only and is not intended for medical or commercial use, and is not intended to replace medical therapy. The device is intended for indoor use in dry rooms. It is not intended for medical, therapeutic, or commercial use.

For safety and CE certification reasons, this product must not be rebuilt or modified in any way. Using the product for purposes other than those described may result in damage to the product. Improper use may also result in hazards such as short circuits, fire, electric shock, etc.

WARNING!

- Danger of improper use!
- Using the device in a manner other than its intended purpose and/or using the device in any other way may involve various risks.
- The device should only be used for its intended purpose.

- Please follow the procedures described in this instruction manual.
- Keep the product away from children.

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or damage due to faulty packaging or transport, please contact the service hotline.

Technical data of the device

Symbol	27727
EAN	5907451382486
Material	plastic
Display	LCD
Power supply	Button battery
Measurement range	1.1–33.3 mmol/L
Measurement range	20–600 mg/dL
Measurement time	about 5 seconds
Blood sample required	approx. 0.5 µl
Result units	mmol/L or mg/dL
Memory	up to 300 results
LO message	Result below 1.1 mmol/L
HI message	Result above 33.3 mmol/L

PREPARING FOR FIRST USE

- Remove the meter and accessories from the packaging.
- Check that the set is complete and that the individual components are not damaged.
- Make sure the battery is properly installed in the meter.
- the date and time if necessary .
- Select the appropriate time format: 12 h or 24 h .
- Check the unit of measurement displayed by the device.
- Before your first measurement, familiarize yourself with the markings on the screen.
- Store test strips in the designated container.
- Do not use the test strip if it is damaged or dirty.
- a new lancet for each puncture .

Startup/Use

- Please remove the packaging materials and dispose of them properly and clean the product and accessories.
- Insert the batteries as marked, paying attention to the correct polarity (+ and -).
- Close the compartment and make sure the battery is seated correctly.

-
- Wash your hands with soap and water.
 - Dry them thoroughly.
 - Prepare your meter, test strip, lancing device and new lancet.
 - Remove one test strip from the container.
 - Close the container with the remaining strips.
 - Place the test strip in the appropriate slot on the meter.
 - Prepare the lancet according to its design.
 - Place a new, unused lancet in it.
 - Prepare your fingertip for pricking.
 - Prick the side of your fingertip.
 - Obtain a small drop of blood.
 - Approximately 0.5 µl of blood is required to perform the measurement .
 - Apply the sample to the appropriate part of the test strip.
 - Do not smear blood across the entire surface of the strip.
 - Once the sample has been taken correctly, wait for the measurement to complete.
 - The result should appear after about 5 seconds .
 - Read the result from the LCD display.
 - Pay attention to the unit of the result – mmol/L or mg/dL .
 - If necessary, mark the measurement as pre-meal, post-meal, or special using the device features.
 - The result can be saved in memory along with the date and time.
 - When finished, remove the used strip.
 - The strip is intended for single use only .
 - Remove the used lancet from the lancing device.
 - Also, do not reuse the lancet .
 - Dispose of used items safely.
 - If the result is unusual or inconsistent with how you feel, take another measurement according to the instructions and, if necessary, consult your doctor.

MARKING MEASUREMENTS

The device allows you to distinguish saved results according to the situation in which the test was performed.

Before Meal – allows you to mark the result taken before eating.

After a meal – allows you to mark the result taken after a meal.

Special marking – allows for additional highlighting of the result in the device memory.

This function helps you organize your saved measurements and more easily distinguish between results taken at different times of the day.

RESULTS MEMORY

- The meter can store up to 300 results .
- Each record can contain the measurement result and the corresponding date and time.
- The device buttons are used to view the saved results.
- The memory function allows you to compare previous measurements.
- When using the memory, pay attention to the date, time and marking of the given result.
- The device's memory is intended to assist in self-monitoring and does not replace medical documentation or interpretation by a specialist.

LO ANNOUNCEMENT

LO message means that the obtained result is below the lower limit of the device's measuring range – 1.1 mmol/L .

- Do not treat the LO message as an exact numerical value.
- Check if the measurement was performed correctly.
- If necessary, repeat the test using a new strip.
- If such a result repeats or disturbing symptoms occur, immediately follow the medical recommendations received from your doctor or contact your healthcare professional .

HI ANNOUNCEMENT

The message HI means that the result is above the upper limit of the device's measuring range – 33.3 mmol/L .

- The HI message should not be interpreted as an exact numerical value.
- Check if the measurement was performed correctly.
- If necessary, repeat the test using a new strip.
- If this result persists or if you experience any unusual symptoms, please contact your doctor or other appropriate healthcare professional .

Battery replacement instructions:

- Please insert the batteries correctly, paying attention to the +/- polarity.
- Only one type of battery should be used.
- Always replace the entire set of batteries.
- Exhausted batteries should be removed from the device and never thrown into fire.
- Never try to charge regular batteries.
- Store unused batteries in their packaging away from metal objects.
- Do not leave children unsupervised.

Cleaning and maintenance

- Remove the test strip from the device before cleaning.
- Wipe the meter casing with a soft, slightly damp cloth.
- Do not immerse the meter in water.
- Do not wash the device under running water.
- Do not allow liquids to enter the test strip slot.
- Do not use abrasive materials.
- Do not use gasoline, solvents or aggressive preparations.
- Do not insert sharp objects into the device.
- Keep the lancing device clean according to its intended use.
- Do not store a used lancet in the lancing device.
- Store test strips in an appropriate container.
- Protect your straps from dirt and moisture.
- Do not touch the strip sampling area with dirty or wet hands.
- Check your blood glucose meter regularly.
- If the housing or display is damaged, discontinue use.

Safety tips and rules

- Before first use, read the instruction manual carefully and keep it for future reference.
- The glucometer should be used only for its intended purpose, i.e. for self-monitoring of glucose concentration in human capillary whole blood .
- The result obtained using a glucometer does not replace a medical consultation, diagnosis or recommendations of medical personnel .
- Do not change your treatment or medication dosage based solely on a single measurement result without appropriate medical advice.
- If the measurement result is unusual, unexpected or does not correspond to the way you feel, check whether the test was performed correctly and, if necessary, repeat the measurement using a new strip.
- If disturbing results persist or disturbing symptoms occur, please contact your doctor or appropriate medical personnel.
- LO message indicates a result below the lower limit of the device's measurement range and should not be interpreted as an exact numerical value.

-
- The message HI indicates a result above the upper limit of the device's measuring range and should not be interpreted as an exact numerical value.
 - Always check the unit of the displayed result – mmol/L or mg/dL – before writing it down or interpreting it.
 - Values given in mmol/L should not be confused with values expressed in mg/dL.
 - Before taking a measurement, wash your hands thoroughly with soap and water and dry them thoroughly.
 - Do not take the measurement with your hands wet, dirty or covered with substances that may contaminate the sample.
 - Use a new, unused test strip for each measurement .
 - The test strip is for single use only.
 - Do not reuse a strip that has already been measured.
 - Do not apply another blood sample to a used strip.
 - Only use test strips compatible with your specific meter model.
 - Do not use strips after their expiration date.
 - Do not use strips that are cracked, deformed, dirty, wet or otherwise damaged.
 - Store test strips in the designated container, protecting them from moisture, dirt and direct sunlight.
 - After removing a strip, immediately close the container with the remaining strips.
 - Do not touch the blood sample area of the strip with wet or dirty fingers.
 - a new lancet for each puncture .
 - The lancet is intended for single use only.
 - Do not reuse a used lancet as this increases the risk of infection and may make the puncture more painful.
 - Do not use one lancet on several people.
 - Do not share your lancing device with other people without following proper hygiene rules.
 - Do not touch the lancet blade with your bare hands.
 - Do not use the lancet if its cover or protective packaging has been damaged.
 - The used lancet should be disposed of in a way that protects other people from accidental needlestick injury.
 - Used test strips and lancets may come into contact with blood and should therefore be treated as potentially biologically contaminated.
 - Wash your hands thoroughly after contact with blood.
 - Do not leave used lancets or strips within the reach of children.
 - Keep the lancing device, lancets, strips and meter out of the reach of small children .
 - Do not allow children to play with the lancing device or lancets.
 - Do not point the lancing device at your face, eyes or other people.
 - Keep the button cell battery out of the reach of children as swallowing it may pose a serious risk to health and life .
 - If a button cell battery is swallowed, seek medical attention immediately.
 - When replacing the batteries, observe the marked polarity.

- Do not short-circuit the battery terminals with metal objects.
- Do not throw the battery into fire, disassemble it or expose it to high temperatures.
- Do not dispose of used batteries with normal municipal waste.
- Protect the meter from water and other liquids.
- Do not immerse the device in water or wash it under running water.
- Do not allow blood, water or cleaning solution to enter the test strip slot.
- Do not insert metal or sharp objects into the strap slot.
- Protect your device from drops, strong impacts and crushing.
- Do not use the meter if the housing, display or strip slot is damaged.
- Do not disassemble the device or modify its electronic components yourself.
- Store your meter and accessories in a clean, dry and safe place , in accordance with the storage conditions specified for the product.

Safety instructions for batteries/rechargeable batteries

- Keep batteries/rechargeable batteries out of the reach of children. If swallowed, seek medical attention immediately!
- Disposable batteries must not be recharged. Batteries/rechargeable batteries should not be short-circuited and/or opened. This may result in overheating, fire, or explosion.
- Never throw batteries/rechargeable batteries into fire or water.
- Batteries/rechargeable batteries should never be subjected to mechanical stress.
- Risk of acid leakage from batteries/accumulators.
- Avoid extreme conditions and temperatures that may affect batteries/accumulators, e.g. radiators/direct sunlight.
- If batteries/rechargeable batteries leak, avoid contact with skin, eyes, and mucous membranes! Immediately flush the affected areas with clean water and seek medical attention!
- Leaking or damaged batteries/rechargeable batteries can cause chemical burns when in contact with skin. Therefore, wear appropriate protective gloves.
- The product has a built-in battery that cannot be replaced by the user.

Warning and information symbols

	<u>READ THE INSTRUCTIONS FOR USE</u> Reading the instructions before use will help you perform the measurement correctly, use the meter properly and reduce the risk of incorrect results or incorrect use of the device .
	<u>TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING</u> The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center. Used packaging material should be delivered to a designated waste disposal site designated by local authorities. For information on recycling options for used products, please contact your local municipality or city office.

	<u>DISPOSAL OF USED ELECTRICAL AND ELECTRONIC EQUIPMENT</u> For environmental reasons, used electrical and electronic products should not be disposed of with regular household waste, but should be disposed of properly. Information on collection points and their opening hours can be obtained from the relevant authorities.
	<u>BIOLOGICAL HAZARD</u> Used test strips, lancets, and other items that come into contact with blood may pose a potential biohazard. Do not reuse them or touch contaminated surfaces. After testing, they should be safely secured and disposed of according to local sharps or biohazard waste management policies.
	<u>CAUTION – SHARP ELEMENTS / STABILITY HAZARD</u> Lancets contain sharp edges that can cause punctures, lacerations, and contact with blood. Exercise extreme caution when inserting, using, and disposing of lancets. Do not touch the blade and do not reuse the lancet. Store the used lancet in a puncture-resistant container and dispose of it according to local regulations.
	This product complies with the requirements of the applicable European and national directives. The product meets European and national requirements for the safety of devices and products.
	This product complies with the requirements of the relevant European and national RoHS directives.
	<u>Improper disposal of batteries/accumulators poses a threat to the environment!</u> Batteries/accumulators should not be disposed of with household waste. They may contain harmful heavy metals and should be treated as special waste. The chemical symbols for heavy metals are: Cd = cadmium, Hg = mercury, Pb = lead. Therefore, used batteries/accumulators should be taken to municipal hazardous waste collection points. The built-in battery must not be disassembled for disposal. The product should be taken to a collection point for used electronics.

We reserve the right to make changes to the text, design and product data without notice.

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